



# FACE SHIELD - PRO

(THIS CONCEPT HAS AN ANATOMICAL CONCEPT THAT PROVIDES GREATER SECURITY AGAINST FLUIDS, HELPING TO BLOCK THE UNCONSCIOUS CONTACT OF THE HANDS WITH THE FACE)



## DATA SHEET

### CONTENT

- Frame (One unit).
- Visor (One unit).
- Tension Strap (One unit).

100% RECYCLABLE PRODUCT



**Use:** Applies to all kinds of activities (Medical, food handling, grocery shopping attendants, banking, drug stores, etc).

## TECHNICAL SPECS



- 100% original Polypropylene shield frame, ergonomic design, it has superior protection and structural reinforcements that guarantee greater user comfort.
- 100% original PVC tension strap, it has different adjustment holes.
- 100% original PET frontal visor, resistant to corrosion, humidity, impact and minor scratches.
- Frontal visor thickness: 0.25 millimetres.
- Net weight: 60 grams.

Light Product



## METHOD OF USE



1. Remove the frame and the tension strap from packaging.
2. Carefully remove the visor from the packaging bag.
3. Adjust the frame with the tension strap at a position of comfort for your head.
4. Remove the frame from your head and install the frontal visor starting with the central locking pins followed by the lateral pins.
5. For the lower shaping of the mask (chin area), fasten the tabs from BELOW in the slots located in the central perforated frame.

## RECOMMENDATIONS



- Clean and sanitize all components with alcohol before assembling the product.
- Use a microfiber cloth to wipe the visor.
- Do not utilize other cleaning and chemical products to disinfect the face shield.
- Avoid excessive manipulation of the face shield.
- Disinfect and sanitize with alcohol the face shield surface.
- Be sure to follow the assembly method in such a way that when making the lower shaping of the mask, there are no exposed edges that could harm health.